




Strategic Business Holdings, Inc.

...Quality Products, Superior Customer Service, & Support

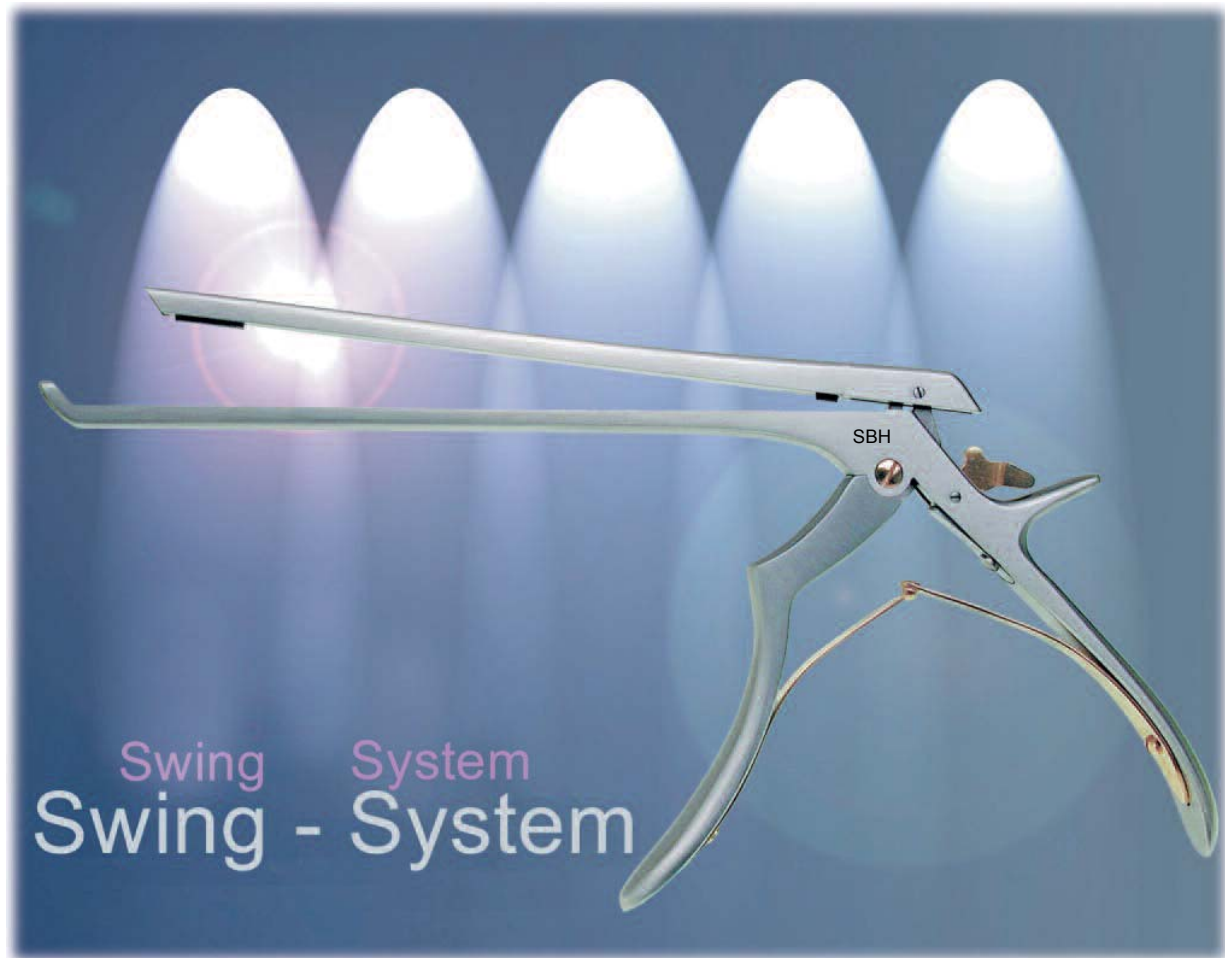

for easy cleaning after use

100% made in Germany

Swing-System
One Piece Detachable Laminectomy Punches


Copyright © SBH ®. 2020 - All rights reserved.



Swing-System One Piece Detachable Laminectomy Punches

Pinzas cortantes para Laminectomía desmontables

- one piece
- easy to disassemble
- easy to clean
- easy to assemble

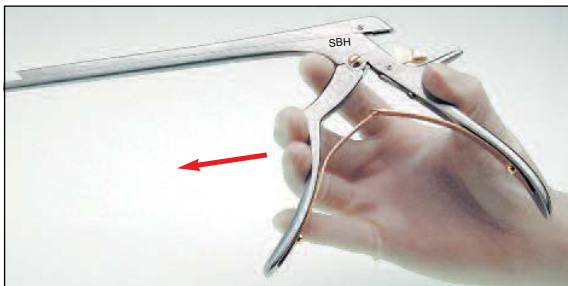

Disassembly

First close the handle ...

than push the lever downwards ...

now open the handle ...


and push downwards at the end of shaft.

- one piece
- easy to disassemble
- easy to clean
- easy to assemble

Assembly




First push shaft parts into one other ...

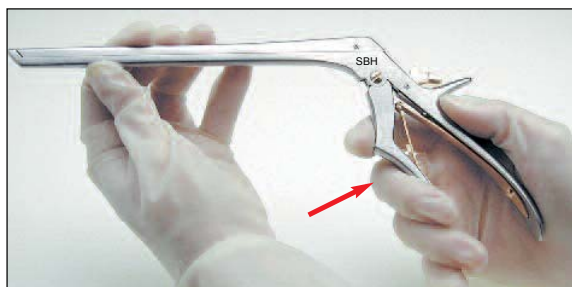


than close the handle ...



now push the lever upwards ...

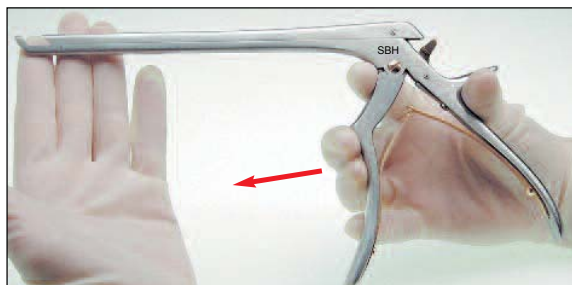

and finally open the handle.


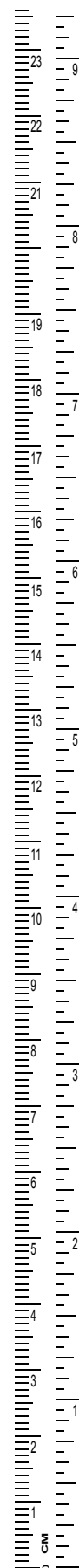




Swing-System One Piece Detachable Laminectomy Punches

Pinzas cortantes para Laminectomía desmontables

Not Actual Size Shown

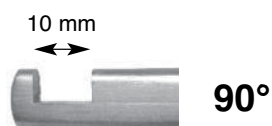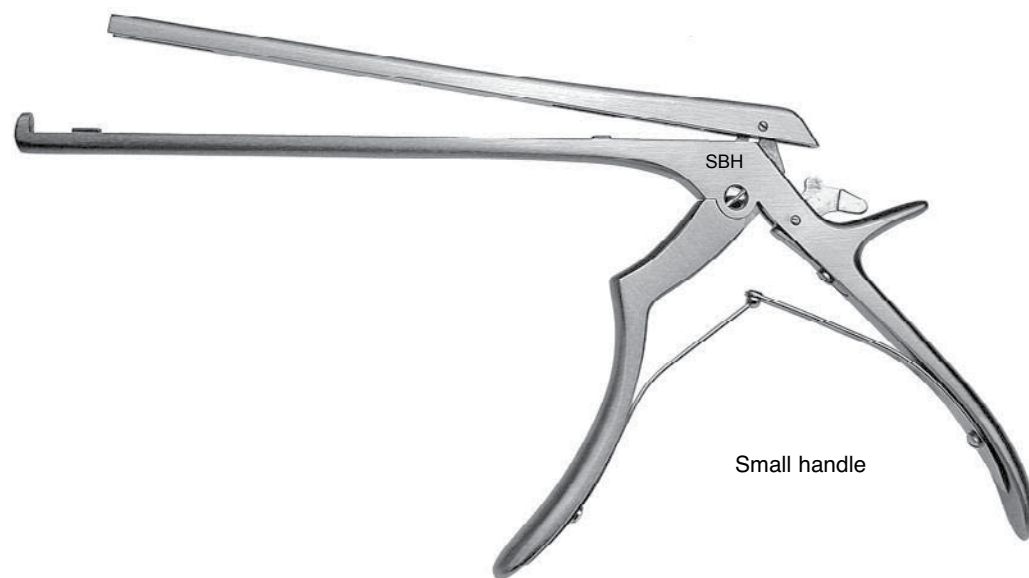

KERRISON

Width of bite	Length of shaft		
	180 mm	200 mm	230 mm
1 mm	S49-051-01	S49-052-01	S49-053-01
2 mm	S49-051-02	S49-052-02	S49-053-02
3 mm	S49-051-03	S49-052-03	S49-053-03
4 mm	S49-051-04	S49-052-04	S49-053-04
5 mm	S49-051-05	S49-052-05	S49-053-05
6 mm	S49-051-06	S49-052-06	S49-053-06



Also available in Ceramic Coating

Swing-System One Piece Detachable Laminectomy Punches

Pinzas cortantes para Laminectomía desmontables




Not Actual Size Shown

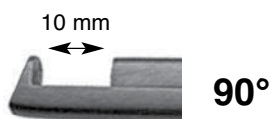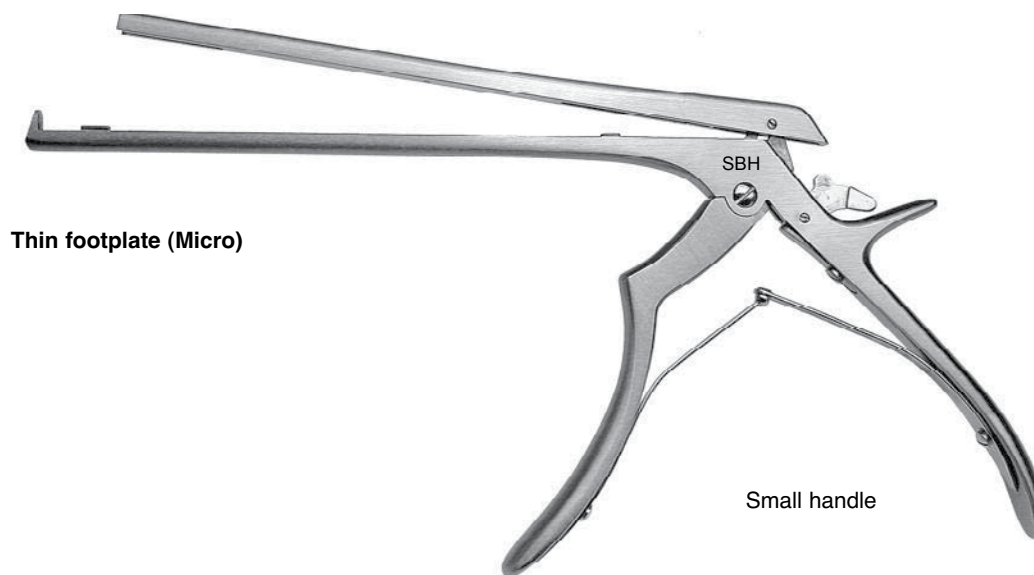

KERRISON

Width of bite	Length of shaft		
	180 mm	200 mm	230 mm
2 mm	S49-051-02T	S49-052-02T	S49-053-02T
3 mm	S49-051-03T	S49-052-03T	S49-053-03T
4 mm	S49-051-04T	S49-052-04T	S49-053-04T
5 mm	S49-051-05T	S49-052-05T	S49-053-05T
6 mm	S49-051-06T	S49-052-06T	S49-053-06T

Also available in Ceramic Coating


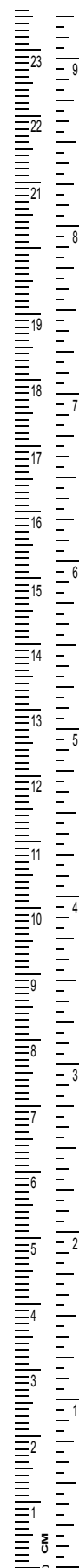




Swing-System One Piece Detachable Laminectomy Punches

Pinzas cortantes para Laminectomía desmontables


Not Actual Size Shown

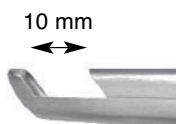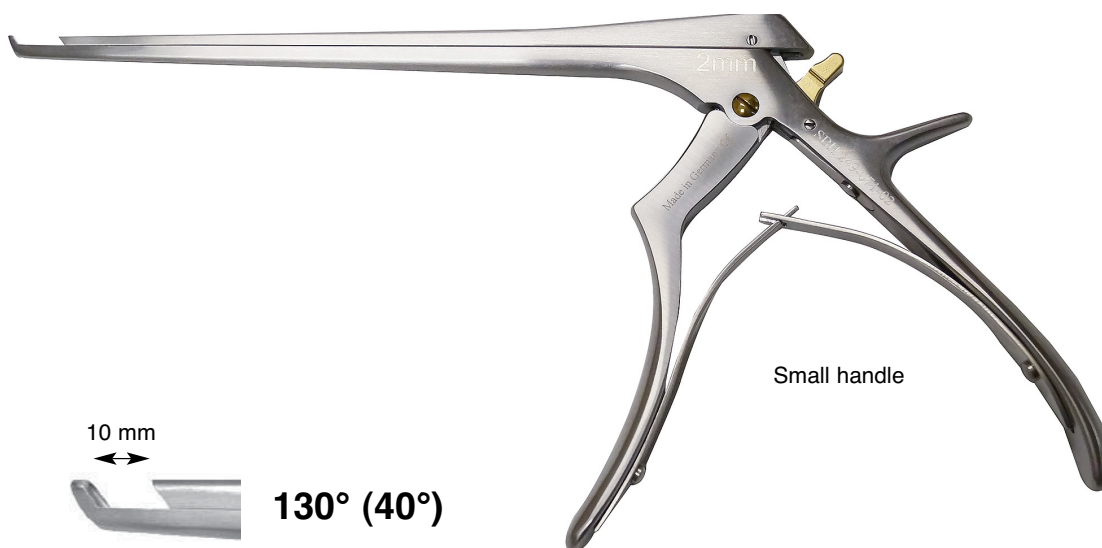

130° (40°)

KERRISON

Width of bite	Length of shaft		
	180 mm	200 mm	230 mm
1 mm	S49-071-01	S49-072-01	S49-073-01
2 mm	S49-071-02	S49-072-02	S49-073-02
3 mm	S49-071-03	S49-072-03	S49-073-03
4 mm	S49-071-04	S49-072-04	S49-073-04
5 mm	S49-071-05	S49-072-05	S49-073-05
6 mm	S49-071-06	S49-072-06	S49-073-06

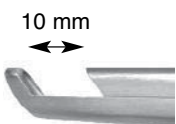

130° (40°) with ejector

KERRISON


Width of bite	Length of shaft		
	180 mm	200 mm	230 mm
2 mm	S49-071-02A	S49-072-02A	S49-073-02A
3 mm	S49-071-03A	S49-072-03A	S49-073-03A
4 mm	S49-071-04A	S49-072-04A	S49-073-04A
5 mm	S49-071-05A	S49-072-05A	S49-073-05A

Swing-System One Piece Detachable Laminectomy Punches

Pinzas cortantes para Laminectomía desmontables



Not Actual Size Shown

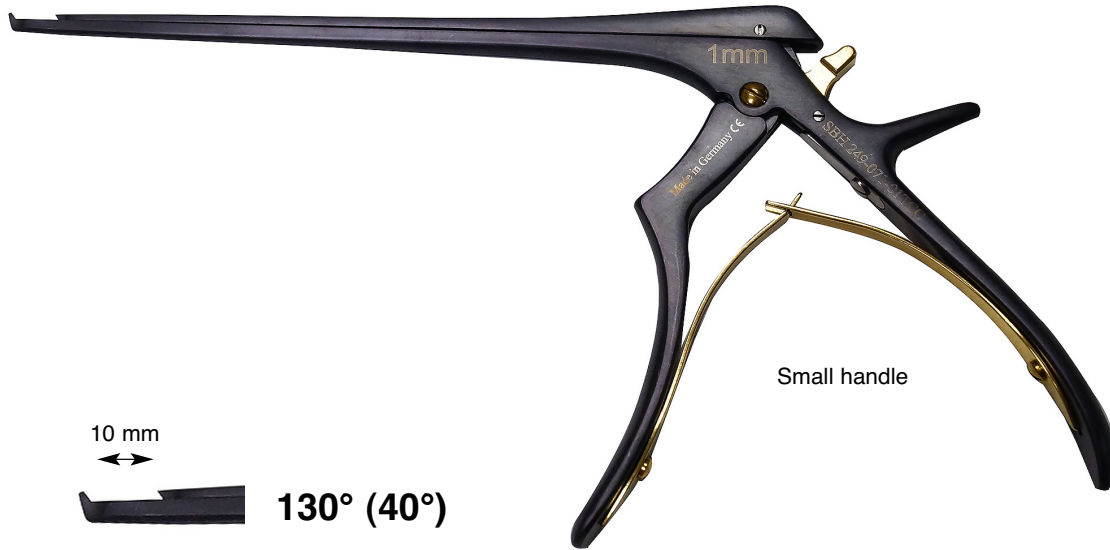

10 mm



130° (40°)

KERRISON

Width of bite	Length of shaft		
	180 mm	200 mm	230 mm
1 mm	S49-071-01CC	S49-072-01CC	S49-073-01CC
2 mm	S49-071-02CC	S49-072-02CC	S49-073-02CC
3 mm	S49-071-03CC	S49-072-03CC	S49-073-03CC
4 mm	S49-071-04CC	S49-072-04CC	S49-073-04CC
5 mm	S49-071-05CC	S49-072-05CC	S49-073-05CC
6 mm	S49-071-06CC	S49-072-06CC	S49-073-06CC

10 mm




130° (40°)

with ejector

KERRISON

Width of bite	Length of shaft		
	180 mm	200 mm	230 mm
2 mm	S49-071-02ACC	S49-072-02ACC	S49-073-02ACC
3 mm	S49-071-03ACC	S49-072-03ACC	S49-073-03ACC
4 mm	S49-071-04ACC	S49-072-04ACC	S49-073-04ACC
5 mm	S49-071-05ACC	S49-072-05ACC	S49-073-05ACC

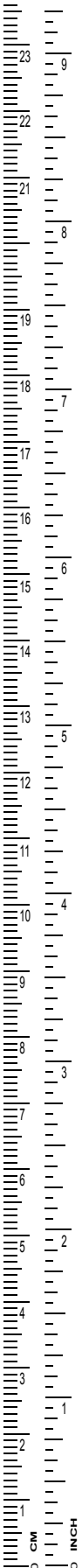




Swing-System One Piece Detachable Laminectomy Punches

Pinzas cortantes para Laminectomía desmontables

Not Actual Size Shown

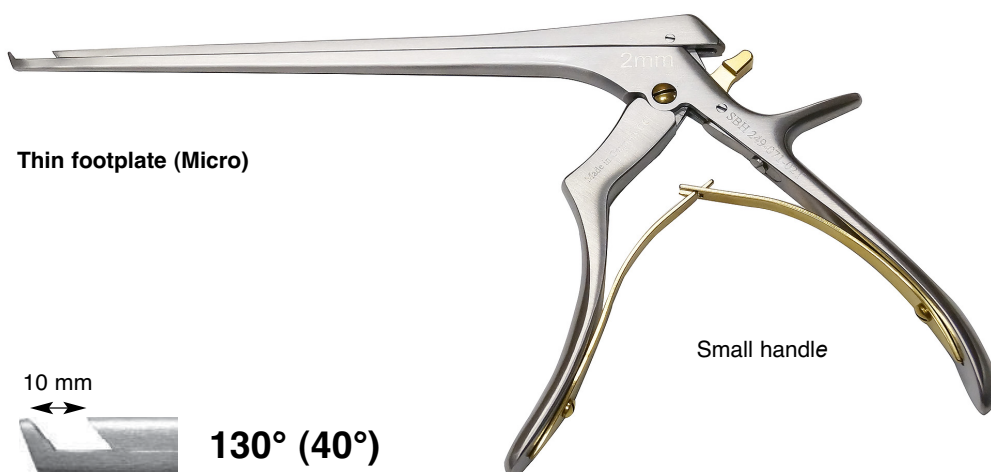

KERRISON

Width of bite	Length of shaft		
	180 mm	200 mm	230 mm
1 mm	S49-071-01T	S49-072-01T	S49-073-01T
2 mm	S49-071-02T	S49-072-02T	S49-073-02T
3 mm	S49-071-03T	S49-072-03T	S49-073-03T
4 mm	S49-071-04T	S49-072-04T	S49-073-04T
5 mm	S49-071-05T	S49-072-05T	S49-073-05T
6 mm	S49-071-06T	S49-072-06T	S49-073-06T



KERRISON


Width of bite	Length of shaft		
	180 mm	200 mm	230 mm
2 mm	S49-071-02AT	S49-072-02AT	S49-073-02AT
3 mm	S49-071-03AT	S49-072-03AT	S49-073-03AT
4 mm	S49-071-04AT	S49-072-04AT	S49-073-04AT
5 mm	S49-071-05AT	S49-072-05AT	S49-073-05AT

Swing-System One Piece Detachable Laminectomy Punches

Pinzas cortantes para Laminectomía desmontables



Not Actual Size Shown

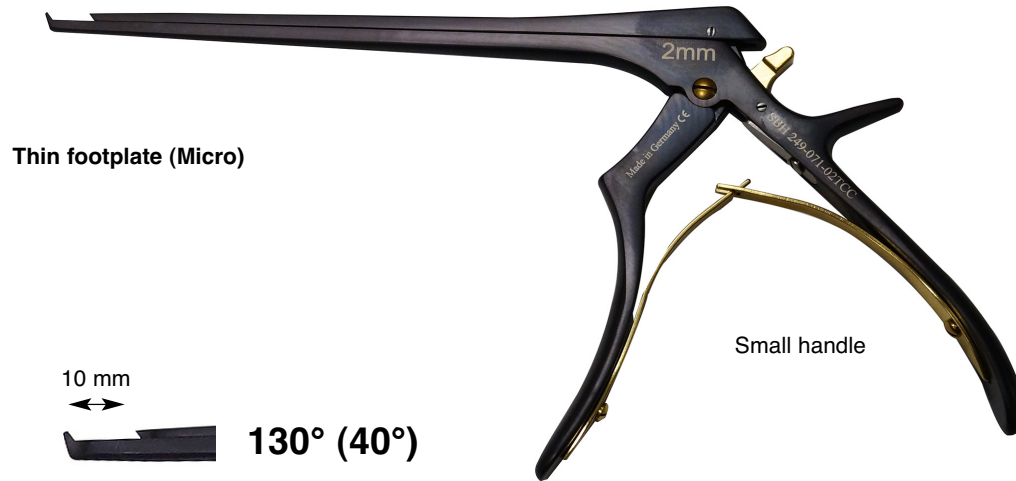

KERRISON

Width of bite	Length of shaft		
	180 mm	200 mm	230 mm
1 mm	S49-071-01TCC	S49-072-01TCC	S49-073-01TCC
2 mm	S49-071-02TCC	S49-072-02TCC	S49-073-02TCC
3 mm	S49-071-03TCC	S49-072-03TCC	S49-073-03TCC
4 mm	S49-071-04TCC	S49-072-04TCC	S49-073-04TCC
5 mm	S49-071-05TCC	S49-072-05TCC	S49-073-05TCC
6 mm	S49-071-06TCC	S49-072-06TCC	S49-073-06TCC

10 mm





130° (40°)


with ejector

KERRISON

Width of bite	Length of shaft		
	180 mm	200 mm	230 mm
2 mm	S49-071-02ATCC	S49-072-02ATCC	S49-073-02ATCC
3 mm	S49-071-03ATCC	S49-072-03ATCC	S49-073-03ATCC
4 mm	S49-071-04ATCC	S49-072-04ATCC	S49-073-04ATCC
5 mm	S49-071-05ATCC	S49-072-05ATCC	S49-073-05ATCC

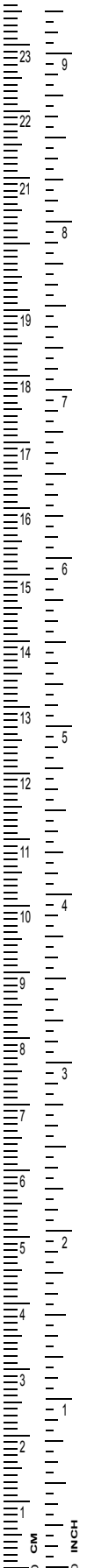

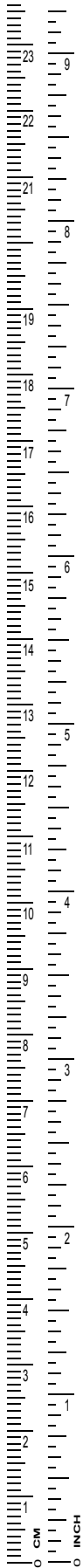[illegible]

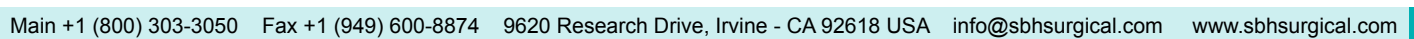



Strategic Business Holdings, Inc.

...Quality Products, Superior Customer Service, & Support

Main +1 (800) 303-3050 Fax +1 (949) 600-8874 9620 Research Drive, Irvine - CA 92618 USA info@sbhsurgical.com www.sbhsurgical.com

Copyright © SBH ®. 2020 - All rights reserved.